

## COVERPLATE for CONFORM® Med. Air Cushion



These individually insertable cover plates are used to clear seating areas (after operations or for open wounds). They can be used with any variant of the 10 cm high CONFORM® Med. air cushions. The pressure-relieving effect of the cushion is not impaired by this.

Cover plates (designation AP) are available for;


AP1 (for one cell)



AP2 (for two cells)



AP4 (for four cells)

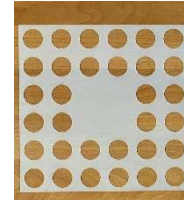

### View and explanation (AP 2 with SKH 4045 (10 cm high, SB 38 x ST 43 cm))



Place the cushion on a flat surface and open all valves.



The cover plate is placed on the airless cushion - in advance, the place where the cells are to be "removed" must be located.



The cells are pulled through the holes in the plate and the plate is to be pressed down to the bottom of the cushion.

Due to the narrowing of the cell at the bottom, the cover plate is fixed without additional aids after the cushion has been filled with air.

Before the cushion is used, it must be covered with the corresponding cover.

**Cleaning** - To clean the cushion, the cover plate can be pulled off after the air has been released from the cushion. Like the cushion, the cover plate can be washed in the washing machine or cleaned with commercially available surface disinfectant.

**Safety note** - Care must be taken that the exposed skin area does not "sag" - this can cause water retention (oedema).



Wolff Kunststoffe GmbH  
Industriestraße 28  
69509 Mörlenbach/Germany  
VAT No.: DE 144458680  
SRN: DE-MF-0000178949

Das company is  
certified according to  
ISO 9001  
ISO 13485

